PREPARATION-, SEPARATION-,
FINISHING- AND POLISHING KITS

Feel the difference!

UK

D

coltène 
whaledent®

ALL-ROUND "RATIO" KIT

(By Dr. N. Hartmann)

Universal preparation and finishing kit

Order No.: 11317

Description:

Page 4

INLAY & CROWN PREPARATION KIT

For the classic preparation of inlays, onlays and crowns

Order No.: 11312

Description:

Page 4

PROFESSIONAL ADHESIVE KIT

(By Prof. Dr. I. Krejci)

Rational preparation kit for adhesive dentistry

Order No.: 11319

Description:

Page 5

CROWN & BRIDGE SEPARATION KIT

For the separation of the hardest alloys in the shortest possible time

Order No.: 11321

Description:

Page 5

AMALGAM PREPARATION KIT

A combination of instruments for the reduction of the tooth; leaves a smooth, perfect cavity margin

Order No.: 11315

Description:

Page 6

AMALGAM REMOVAL KIT

A combination of diamonds and tungsten carbide burs for fast and safe amalgam removal

Order No.: 11320

Description:

Page 6

UNIVERSAL SHAPENING AND FINISHING KIT

Microfine finishing diamonds for an optimum surface of the restoration

Order No.: 11314

Description:

Page 7

"PARO" PERIODONTOLOGY KIT


12 diamonds for the improved treatment of the root surface

Order No.: 11322

Description:

Page 7

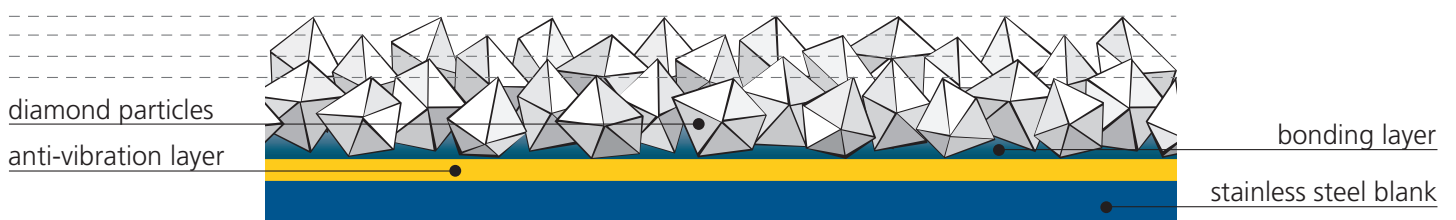DIATECH's Exclusive **Multilayer** Diamond Coating

Feel the difference!

COMPOSITE POLISHING KIT

The two-step polishing system creating a smooth and shiny surface

Order No.: 13001
Description:

Page 8

PORCELAIN POLISHING KIT

For atraumatic polishing of modern porcelains

Order No.: 13003
Description:

Page 8

AMALGAM POLISHING KIT

An assortment of popular silicone polishers for a smooth surface and high lustre

Order No.: 13002
Description:

Page 9


"BRUSHINE" UNIVERSAL POLISHING KIT

For high-shine polishing on occlusal surfaces and interdental spaces within seconds

Order No.: 7010
Description:

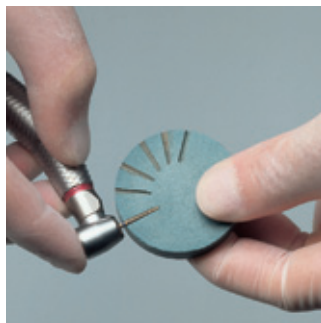Page 9


"PULIMANT" DIAMOND CLEANING BLOCK

Optimizes the durability of your valuable diamonds

Order No.: 11346
Description:

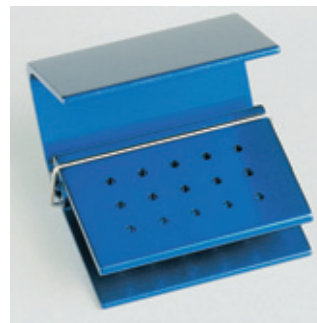Page 10

AUTOCLAVABLE BUR BLOCKS

The perfect manager of your instruments with protection against falling out during sterilization

Order No.: 70005 / 70013 / 70019
Description:

Page 10

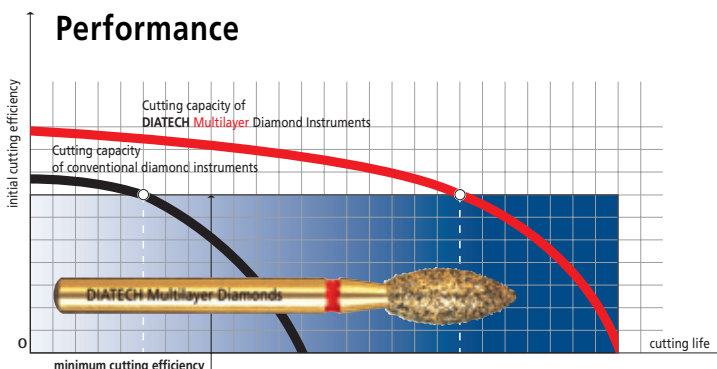
DIATECH Diamond Instruments for Dental Practice

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DIATECH Multilayer diamonds

are multiple layered in an exclusive manufacturing process using the most advanced technology, while the diamond instruments you used in the past probably had just one layer of diamond. Multiple layered diamonds are provided with more diamond on different levels and in various layers. New diamond grits with sharp edges will, therefore, take the place of broken off grains or grains separated from the plating during normal use.

Multilayer DIATECH diamonds have an extended life and cut faster.

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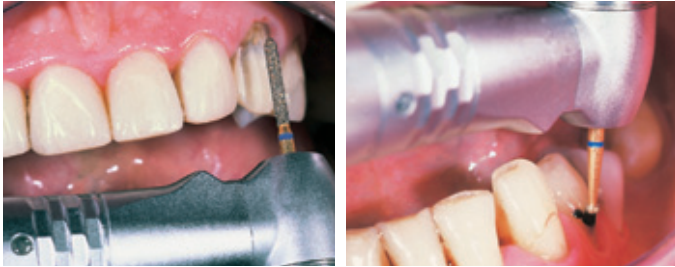
(By Dr. N. Hartmann)

Contains: 18 FG Multilayer Diamonds

The non plus ultra all-round kit in the DIATECH range. If you have never worked with DIATECH Multilayer diamonds, take a close look at their impressive durability, high efficiency and quality. Should you already be an enthusiastic user of DIATECH Multilayer diamonds, the "Ratio" kit provides you with 18 selected Multilayer FG diamonds with the well known high DIATECH quality and at an unbeatable price.

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INLAY & CROWN PREPARATION KIT

Contains: 14 FG Multilayer Diamonds

Everything depends on the geometry of the preparation being correct. Make use of this ingenious selection of 8 Multilayer FG preparation diamonds and 6 Multilayer FG finishing diamonds to perfect your inlay-, onlay- and crown preparations systematically and with the correct geometry. The outcome: Minimal loss of tooth structure and optimum retention. The best preconditions for producing long lasting restorations, for your patients' benefit.

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Pure adhesive dentistry! When using this instrument kit - developed by one of the leading experts of adhesive techniques - you have modern adhesive restoration techniques fully under control. This kit contains 12 Multilayer FG diamonds and 1 FG/3 RA tungsten carbide burs and can be used for removing old restorations and excavating caries as well as preparing, finishing and trimming adhesive fillings, inlays, onlays, crowns, laminate veneers and bridges.

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CROWN & BRIDGE SEPARATION KIT

- Contains: 1 FG Multilayer Diamond
4 FG Tungsten Carbide Burs

Nothing could be more time saving! These 3 cross-cut burs and 1 tungsten carbide bur plus the coarse Multilayer diamond cylinder separate even the hardest of alloys in the shortest possible time. Even if these instruments are used repeatedly, their efficiency hardly drops. Try them!

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AMALGAM PREPARATION KIT

Contains: 8 FG Multilayer Diamonds

This instrument kit combines the advantages of the well-known Multilayer diamond technology with specific instrument shapes for macroretentive cavity preparations.

The 4 Multilayer preparation diamonds with a 125 micron grit size help you to reduce the tooth structure efficiently and are long-lasting. The 4 Multilayer finishing diamonds with a 25 micron grit size create smooth, crisp cavity margins which are essential for optimal marginal adaptation of your amalgam fillings.

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AMALGAM REMOVAL KIT

Contains: 4 FG Multilayer Diamonds
6 FG Tungsten Carbide Burs

Thanks to prophylaxis, new fillings are becoming less and less common but you are now confronted with defective amalgam fillings daily. This instrument kit will not let you down. 4 coarse grit Multilayer diamonds, 2 cross-cut tungsten carbide burs and 4 tungsten carbide round burs turn removing large and small amalgam fillings, including excavating secondary caries, into child's play.

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Feel the difference!

UNIVERSAL SHAPENING AND FINISHING KIT

Contains: 14 FG Multilayer Diamonds

This finishing kit comprises 8 Multilayer FG diamonds for contouring and 6 Multilayer FG diamonds for finishing amalgam, composite, compomer, porcelain, glassionomer and gold restorations. DIATECH fine grit Multilayer diamonds are highly efficient and will not harm the surface of the restoration nor the adjacent tooth structure.

We owe it to your high grade restorations!

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"PARO" PERIODONTOLOGY KIT

Contains: 12 RA Multilayer Diamonds

Calculus-free tooth roots are an important factor for your patients' periodontal health. DIATECH helps you by producing a set of diamond burs which are ideal for power-assisted root surface preparation.

The 65 micron burs are for contouring the tooth. The 45 micron diamond burs remove calculus easily. The 15 micron burs smooth the root surface quickly yet conserve tissue. This type of periodontology is fun!

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COMPOSITE POLISHING KIT

Contains: 16 Silicone Polishers

Smooth surfaces offer the best protection against plaque accumulation and discolouration. You know that this is especially important for composite materials. These recently developed DIATECH silicone polishers with medically tested silicone create a smooth, shiny surface on composites - even on tricky fine hybrid composites.

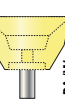
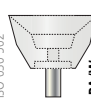
The kit comprises 8 yellow RA instruments for prepolishing and 8 light grey RA instruments for high lustre polishing.

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PORCELAIN POLISHING KIT






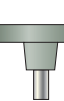
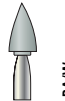
Contains: 15 Silicone Polishers

The surfaces of modern porcelains and glass-ceramics must be polished atraumatically. DIATECH has developed special RA silicone polishers for this purpose. The dark green and dark grey cups, wheels, knife-edge wheels and points are used for prepolishing. A pink point, knife-edge wheel and cup are available for high lustre polishing.

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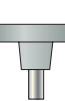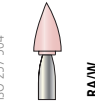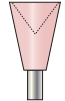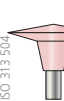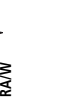
AMALGAM POLISHING KIT

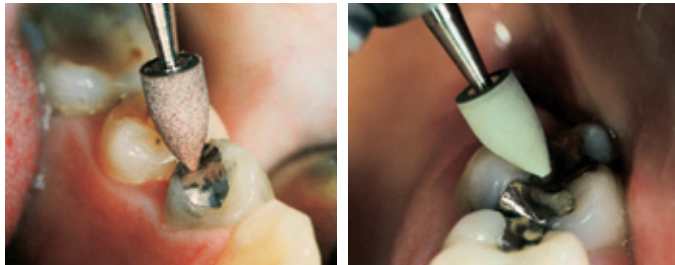Contains: 16 Silicone Polishers







Well-tried and tested polishing points, and cups, made of a new type of silicone mixture which was developed with medically tested silicone by DIATECH specifically for use on amalgam surfaces.

8 brown RA instruments for prepolishing and 8 green RA instruments for creating an unbeatable high lustre - all combined in one polishing kit for you.

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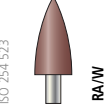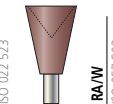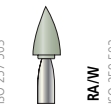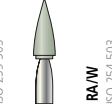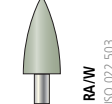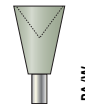"BRUSHINE" UNIVERSAL POLISHING KIT

Contains: 6 "Brushine" Polishing Brushes

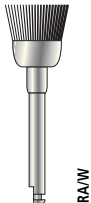
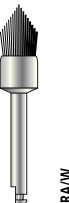
"Brushine" is the start of a new polishing age for dental restorations. DIATECH integrated fine grit silicon carbide polishing powder into thin, smooth bristles so that no polishing paste is required. Due to the three different shapes, you can easily polish those areas of restorations which are difficult to access, such as fissures and interdental spaces, within seconds and without flattening the surface unnaturally. For use by both dentists and dental technicians.

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"PULIMANT" DIAMOND CLEANING BLOCK

Professional care optimizes the durability of your valuable DIATECH diamonds and guarantees that they will remain highly efficient even after repeated, intense use.

DIATECH supplies the Diamond Cleaning Block for this purpose:

Place the diamond instrument flat on the "Pulimant", run it at medium speed and press it gently into the Block - that's it.

Slurry and remnants of fillings are removed reliably from the surface of the instrument and the diamond grit "bites" properly again.

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Order No.:
11346

AUTOCLAVABLE BUR BLOCKS

Put a stop to chaotic bur organization! These small, anodized blocks offer a handy solution. The lid prevents the burs from falling out during transport and storage.

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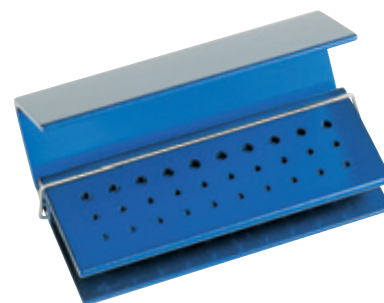
Recommended Hygiene Procedures for Dental Practices

1. General

Protection against infection is especially important to patients and surgery staff. Disinfection and/or sterilization of the instruments used are an absolutely essential part of a safe preparation technique.

The instruments must be sterilized prior to use.

2. Cleaning, sterilizing and storing

After use, instruments must be immersed immediately in a tank containing suitable cleaning agent/ disinfectant for 60 minutes (20 minutes for tungsten carbide burs). Disinfectants contain some chemicals which may corrode tungsten carbide burs. Use a disinfectant/cleaning agent containing a corrosion inhibitor. The cleaning agent/disinfectant manufacturer's instructions must be heeded.

- Strong acidic and alkaline disinfectants may adversely affect the instruments. Strong chemicals attack the instruments.
- The burs should not be left in the disinfectant for an extended period (e.g. over night).
- If necessary, clean by mechanical means in advance (brush the working head with a brush with nylon or brass bristles).
- Check for any remains and, if necessary, clean again.
- The instruments may be cleaned ultrasonically. The use of bur stands or holders is advisable as they prevent the instruments, from vibrating against each other and becoming damaged.
- Sterilize the instruments in a chemidave or autoclave or in hot air.

Hints:

- Avoid temperatures exceeding 190° C.
- Tungsten carbide burs must be completely dry after heat sterilization. If this is not the case, dry them again - always keep them in a dry place.
- Sterilizers must be used in accordance with the manufacturer's instructions.
- The instruments should then be stored in alcohol or ultra-violet surroundings until further use.
- For long term storage, keep instruments in stands or containers in a dry place, protected against dust and acidic vapours.

Please note:

To prevent spreading of infectious germs, instruments contaminated with blood should be disposed of carefully.

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Recommendations for Use and Safe Operation in Dental Practices

1. General

To prevent harm to the residual tooth structure as well as the tissues and teeth contiguous to the site, provide for sufficient cooling, technically, clinically and hygienically optimum instruments and materials as well as optimum instrument speed and only exert the necessary amount of pressure. The service-life of the individual instruments depends on compliance with the following recommendations for use and safe operation.

2. Recommendations for Use and Safe Operation

The ergonomic principles must be taken into account during the working procedure. The handpiece (high-speed handpieces, contra-angles) must be in perfect working order. The rotary instrument must be fully inserted and gripped in the high speed handpiece or contra-angle with the utmost of care.

Select the correct speed to ensure optimum performance and service-life. Operate within the recommended speed ranges and heed the hints covering maximum permissible speeds.

Before placing the instrument in contact with the site being prepared, check that it is rotating at the optimum speed and that the cooling system is functioning properly. To prevent harm to the natural teeth and fillings, adequate amounts of water must be sprayed onto the diamond instruments and tungsten carbide burs to cool them properly. We recommend high-speed handpieces with three jets which spray the coolant along the entire length of the instrument's cutting surface. At least 50 ml/min. of coolant must be provided. Extra-long diamonds and tungsten carbide burs (total length exceeding 21 mm) and those with a head exceeding 2 mm in diameter require additional cooling.

Prepare carefully and without exerting pressure. Select the pressure (0.3 – 2 N (30 – 200 g)) to ensure that the speed is not reduced noticeably. The pressure exerted on the rotary instrument is a decisive factor for the working procedure and the successful outcome of the preparation. Ineffective coolant application (diverted spray, inaccurate techniques) may adversely affect the final results.

Under no circumstances should the instrument be allowed to jam.

Only use impeccable instruments. Instruments which are blunt, damaged, bent or no longer concentric, must be segregated immediately and disposed of carefully.

Whenever possible, the instruments should be stored in their original packaging to enable them to be identified and traced.

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Coltène AG